



ProFront™

Protective visor with head
band

User manual



USER MANUAL IN ORIGINAL

Version 4

Andreas Wilhelmsson

Pamex Plastic AB

2020-06-05

Contents

GENERAL	2
Purpose	2
Basic requirements.....	2
MANUFACTURER INFORMATION.....	2
PRODUCT DESCRIPTION	3
Intended use	3
Product guarantee	3
Product limitations.....	3
DESCRIPTION OF USAGE	5
How to assemble	5
Care and maintenance instructions.....	7
Storage instructions.....	8
SPARE PARTS AND ACCESSORIES	9
RECYCLING.....	9
OTHER	9

General



This product must only be used in accordance with the instructions stated in this manual.

Purpose

The purpose of this user manual is to provide an understanding of how the equipment should be used and an awareness of potential risks.

Basic requirements

These instructions must first be read and understood for the user to be able to benefit from and the use equipment properly.

The user manual should be kept available for all users of the product.

Manufacturer information

Manufacturer: Pamex Plastic AB
Visiting Tallberga 4,
address: 333 73 Bredaryd,
Sweden
Email: info@pamex.se
Telephone: +46 (0)371 -701 85



Product description

Product name: ProFront™

Product description: Protective visor

Article number: 200404

The product is CE marked according to the PPE Regulation (EU) 2016/425 Personal Protective Equipment and meets all the requirements of the method/standard:

SS EN 166:2001

The product is tested and certified by:

RISE Research Institutes of Sweden AB

Box 857

501 15 BORÅS

Sweden

Notified body number 0402

The EU declaration for this product can be found at www.pamex.se/doc

Intended use

This product is intended for protection against droplets and splashes of liquid when testing and working with patients/consumers within health and medical services.

Product guarantee

The product has a one-year guarantee from the date it is first removed from its packaging and assembled. Expected shelf life is approximately 15 months and should not be used after this time after manufacturing date.

Product limitations



PLEASE NOTE:

Do not leave the visor in direct sunlight as it affected by prolonged exposure to UV rays.



WARNING!

Damaged or scratched visors must be replaced.

Description of usage

How to assemble

The following describes how to assemble and use the ProFront™ protective visor correctly:


Step 1:

Remove the head band and the visor from the packaging (*see picture to the right*).




Step 2:

Attach the middle hole on the visor to the hook on the middle of the head band (*see picture to the right*).

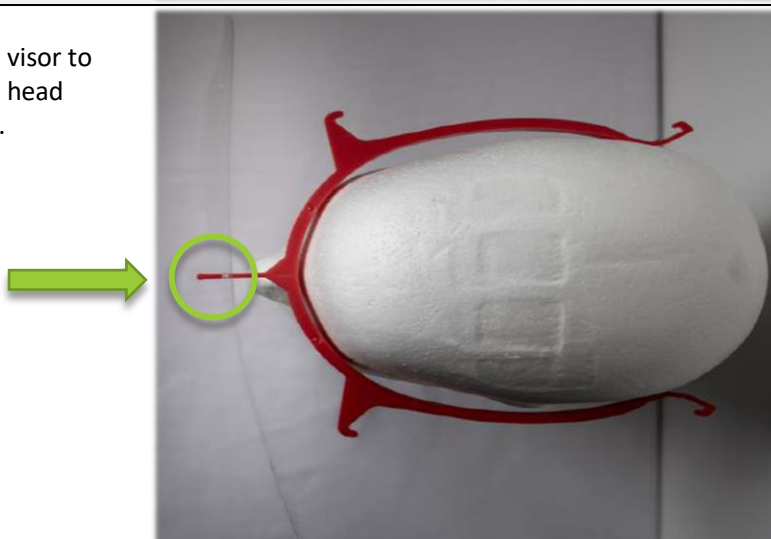


Step 3:

Next, bend the visor and attach each of the outer holes to their hooks on the headband (*see picture to the right*).

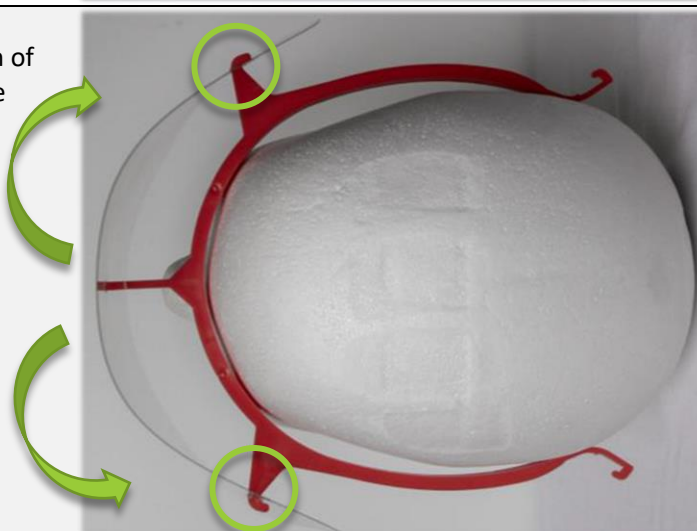

Step 4:

Finally, put the ProFront™ protective visor on your head. The head band should be positioned directly above your eyebrows. Adjust the angle of the visor by moving the rear part of the head band up or down for the best fit (*see picture to the right*).

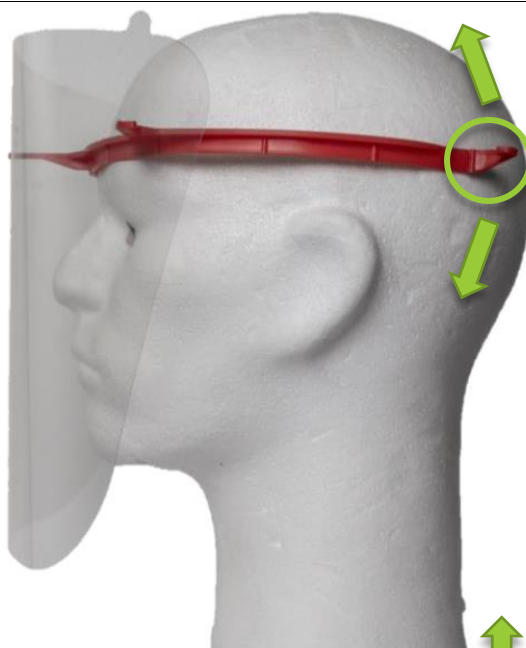

PLEASE NOTE: Always check that the ProFront™ protective visor is correctly positioned before starting work.




Step 5:

To remove the visor without the risk of getting the contagion on your hands:

Take hold the rear hooks, pull them carefully outwards and remove the visor in a forward direction (*see picture to the right*).

Thereafter follow conventional hygiene routines such as carefully washing and/or disinfecting your hands afterwards.

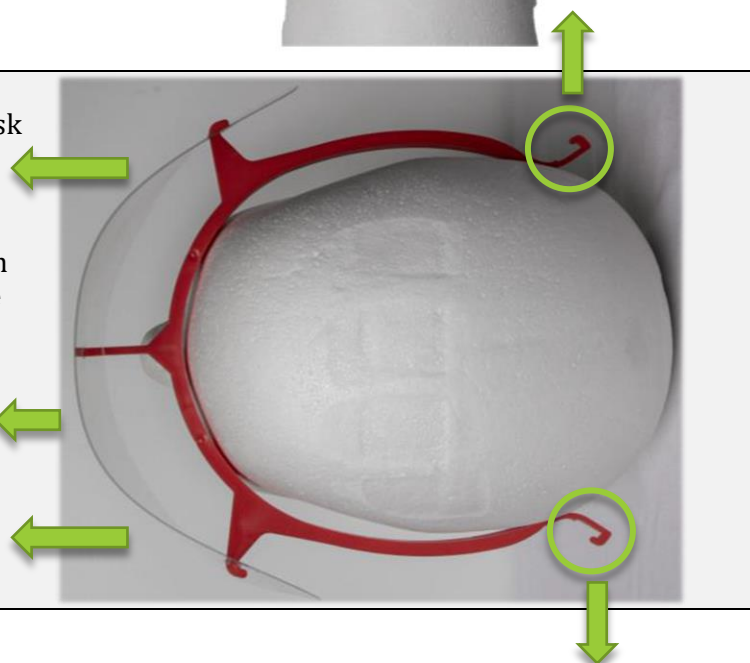

Care and maintenance instructions




PLEASE NOTE:


The ProFront™ protective visor is not intended for long-term usage, but it can be re-used provided that the visor has adequate visibility and has not in any way been damaged.

Maintain and re-use the ProFront™ protective visor as follows:


1. Disinfect your hands with hand sanitizer.

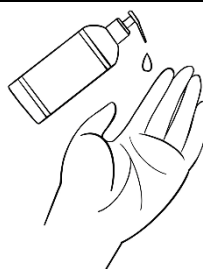


2. Put on a pair of protective gloves.

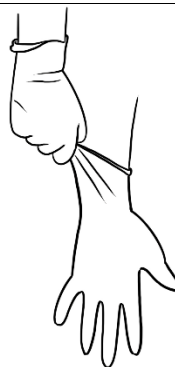


3. Disinfect the visor and head band with a disinfectant that contains a surfactant.

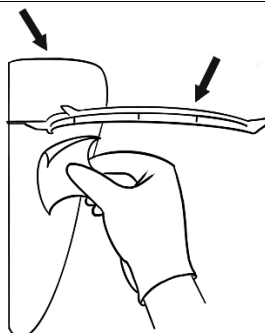

4. Hang the visor up or place it so that it can dry off, alternatively, dry it with a sterile single-use wipe.

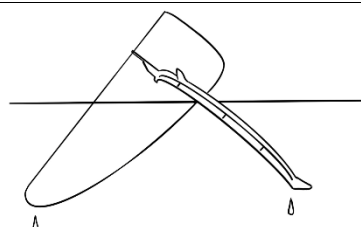


5. Store the dry disinfected visor in a clean plastic bag until it is next used.

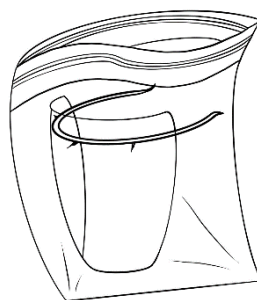

6. Remove the protective gloves.



7. Finally, disinfect your hands with hand sanitizer.

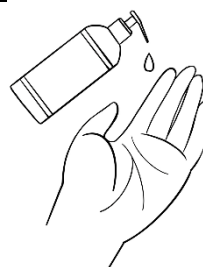

Delivery instructions

ProFront™ protective visor comes in boxes of 200 complete visors. Keep the visors in their packaging until they are to be used. One (1) user manual is packed in each box and it must be kept available for all users of the visors.



Storage instructions

- Visors should be stored carefully to avoid any risk of any external damage.
- Avoid storing the visors in direct sunlight.

Spare parts and accessories

Head band: 2-200404-XX

Visor: 1-00404

Recycling

Once the visor has been used, it should be dismantled in reverse order, per the earlier description for assembly.

The visor and the head band should be recycled as hard plastic, according to applicable local recycling rules.

Other

Pamex would like to take this opportunity to thank you for choosing our product. We are proud to be able to contribute to a safer world.